

Settings

Parameter	Modes			Default	Range	Description
	AutoSet	APAP	CPAP			
Ramp	✓	✓	✓	20 minutes	–	Orange icon shows that device is ramping up.
Set pressure (CPAP)			✓	9 cm H ₂ O	4–20 cm H ₂ O, 0.2 cm H ₂ O increments	Sets the fixed treatment pressure.
Max pressure	✓	✓		20 cm H ₂ O	Min–20 cm H ₂ O, 0.2 cm H ₂ O increments	Sets the upper limit of treatment pressure.
Min pressure	✓	✓		4 cm H ₂ O	4–Max cm H ₂ O, 0.2 cm H ₂ O increments	Sets the lower limit of treatment pressure.
Max ramp	✓	✓	✓	45 minutes	Off–45 minutes, 5-minute increments	Limits the ramp times the patient may select.
Start pressure	✓	✓	✓	4 cm H ₂ O	4 cm H ₂ O, 0.2 cm H ₂ O increments	Sets the pressure at the start of ramp, up to fixed treatment pressure (CPAP) or minimum treatment pressure (AutoSet mode).
EPR	✓	✓	✓	Off	Off / Ramp Only / Full Time	Sets when EPR is to occur.
EPR level	✓	✓	✓	Off	Off / 1 / 2 / 3 / Patient	Sets the EPR value.
Mask	✓	✓	✓	Pillows	Pillows / Full Face / Nasal / Nasal Ultra	Selects the type of mask used by the patient.
Tube type	✓	✓	✓	SlimLine	SlimLine / Standard	Selects the type of air tubing used by the patient.
Climate Control	✓	✓	✓	Auto	Auto / Manual / Patient	Sets the type of Climate Control.
Sleep quality	✓	✓	✓	Usage	On / Usage	Sets Sleep Quality to Usage or On.
SmartStart	✓		✓	Off	On / Off / Patient	Enables or disables the SmartStart feature.
EPR inhale	✓	✓	✓	Med	Med / Fast	Sets the inhalation type.

See the Clinical Guide for full details.

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.

Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK.

See www.resmed.com for other ResMed locations worldwide.

Protected by patents: AU 691200, AU 697652, AU 702820, AU 709279, AU 724589, AU 730844, AU 736723, AU 750095, AU 750761, AU 764761, AU 779327, AU 2007252223, EP 0651971, EP 0661071, EP 0920845, EP 0927538, EP 0927423, EP 1028769, EP 1126893, EP 1144036, EP 1502618, JP 3580776, JP 3778797, NZ 504595, NZ 570415, US 5245995, US 5704345, US 6029665, US 6138675, US 6240921, US 6363933, US 6367474, US 6425395, US 6502572, US 6591834, US 6675797, US 6745768, US 6817361, US 6988498, US 7040317, US 7100608, US 7320320, US 7537010. Other patents pending.

Protected by design registrations: AU 329025, AU 330142, CN 200830346526.0, EU 1067417, JP 1377401, JP 1377402, JP 1377403, NZ 411762, NZ 413090. Design registrations pending.

S9 is a trademark of ResMed Ltd and is registered in U.S. Patent and Trademark Office.

© 2010 ResMed Ltd. 368248/2 2010-06

RESMED

AutoSet™ / Elite™ / Escape Auto™ / Escape™ POSITIVE AIRWAY PRESSURE DEVICES

Quick Setup Guide for Clinicians

English



Operating information

The following table lists the operating modes available on the S9 Series.

Mode	S9 AutoSet	S9 Elite	S9 Escape Auto	S9 Escape
CPAP mode Provides fixed treatment pressure.	✓	✓	✓	✓
AutoSet mode Automatically adjusts pressure in response to inspiratory flow limitation, snore and apnea.	✓			
APAP mode Automatically adjusts pressure in response to inspiratory flow limitation, snore and apnea but only treats apneas up to 10 cm H ₂ O and does not differentiate between different types of apneas.			✓	

Navigating the menus


In general, to navigate the menus:



1. Turn until the parameter you require is displayed in blue.
2. Press . The selection is highlighted in orange.
3. Turn until you see the setting that you require.
4. Press to confirm your choice. The screen returns to blue.




Home menu

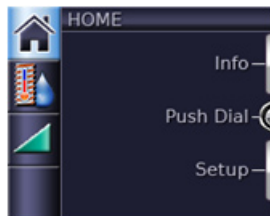

The Home menu shows you and your patient what features are currently activated, and the accessories that are connected to the device.

Treatment screens

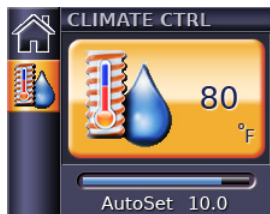


Depending on how the system has been configured, you will see a different screen when the device is running. This example shows:

- ✓ H5i humidifier
- ✓ ClimateLine heated air tube
- ✓ Climate Control – Auto



Standard Info menu

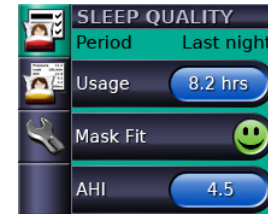

The Standard Info menu provides information about compliance, therapy and settings. Patients can check their sleep quality, sleep report and service information.

Note: Mask Fit is not available in the S9 Escape and S9 Escape Auto. AHI is not available in the S9 Escape.




Advanced Info menu (Hold for 3 sec)



The Advanced Info menu provides patients with additional therapy settings and compliance information.



Patient Setup menu

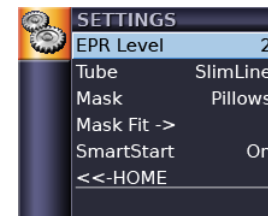


Only settings relevant to the patient are displayed in the Patient Setup menu.

Depending on how the device has been customized via the Clinical Setup menu, the following screens can be viewed: EPR Level; Tube; Climate Ctrl; Mask; Mask Fit; and SmartStart.

Note: SmartStart and Mask Fit are only available in the S9 AutoSet and S9 Elite.



Clinical Setup menu (Hold for 3 sec)

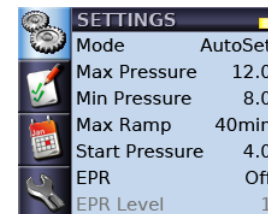

There are four screens available from the Clinical Setup menu: Settings; Options; Reminders; and Configuration.

Clinical menus are identified by the yellow open lock shown in the top right corner.